
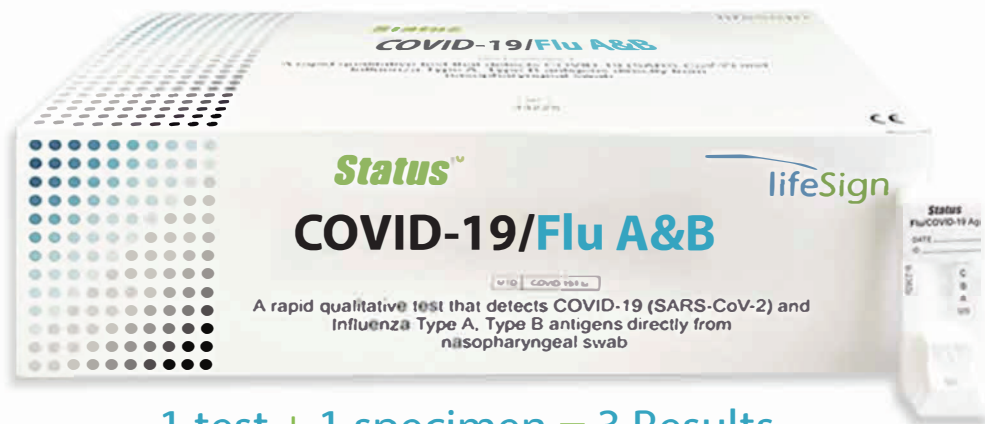


**Status™**

# COVID-19/Flu A&B

A Rapid Immunoassay for the Simultaneous Direct Detection and Differential Diagnosis of SARS-CoV-2, Influenza Type A and Type B Antigen from anterior nasal and nasopharyngeal swab specimens

Infections with these viruses may present similar symptoms. Can you tell them apart? **We Can.**

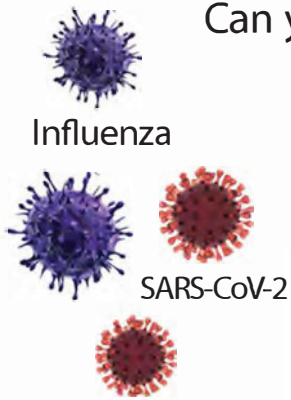


Learn More

1 test + 1 specimen = 3 Results

- **COVID-19 - Anterior Nasal - Sensitivity 93.8%, Specificity 100%**  
**Nasopharyngeal - Sensitivity 93.1%, Specificity 100%**
- **Flu A - Sensitivity 91.4%, Specificity 95.7%**
- **Flu B - Sensitivity 87.6%, Specificity 95.9%**
- **FDA Emergency Use Authorization (EUA)**
- **Visually read in 15 minutes**
- **Flocked swabs for superior specimen collection and patient comfort**


ORDERING INFORMATION		
PRODUCT	# OF TESTS	ITEM #
Status™ Covid-19/Flu A&B	25	33225

 Proudly made in the USA  
For more information or to place an order, contact




☎ 248-781-8700    ✉ info@elproviders.com  
☎ 1-877-745-9943    🌐 www.elproviders.com

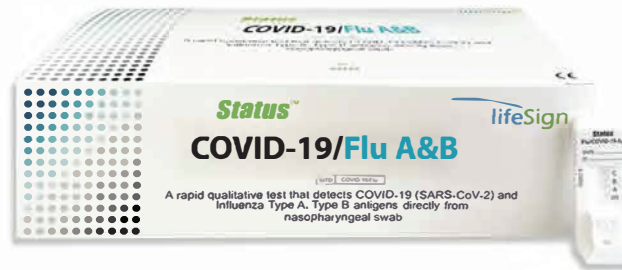


A PBM Group Company  
85 Orchard Road • Skillman • New Jersey • 08558  
800.526.2125 • 732.246.3366 • Fax: 732.246.0570 • www.lifesignmed.com



## Easy 3 Step Procedure
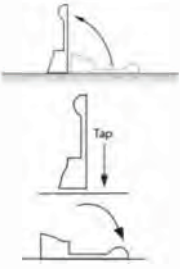
<b>1</b> 	<p>squeeze</p> <p>Tear the tab off the Extraction Reagent Capsule and dispense entire contents into the Extraction Well.</p>
<b>2</b> 	<p>Insert the specimen swab in the Swab Stand.</p> <ul style="list-style-type: none"> <li>• Rotate swab 3 times to mix the specimen.</li> <li>• Let stand 1 minute.</li> <li>• Rotate swab 3 times again and discard the swab.</li> </ul>
<b>3</b> 	<p>Raise the device upright and let stand 1-2 seconds.</p> <p>Gently tap the device to ensure the liquid flows into the hole.</p> <p>Lay the device back down.</p> <p style="text-align: center;"><b>Set a timer for 15 minutes.</b></p>
<b>4</b>	<p><b>Read test results at 15 minutes.</b></p> <p><b>NOTE: False positive or false negative results can occur if the test is not read between 15 and 20 minutes.</b></p>

- Refer to our website at [www.lifesignmed.com](http://www.lifesignmed.com) for complete test procedure details, training video and package insert.
- Status™ COVID-19/FLU A&B is an antigen test for use with anterior nasal and nasopharyngeal swab specimens.
- Status™ COVID-19/FLU A&B test has been granted emergency use authorization (EUA) by the FDA for use at the point-of-care setting. During the duration of the emergency this test can be performed in a patient care setting that is operating under a CLIA Waiver, Certificate of Compliance or Certificate of Accreditation.

 Proudly made in the USA

For more information or to place an order, contact

 248-781-8700   
  [info@elproviders.com](mailto:info@elproviders.com)  
 1-877-745-9943   
  [www.elproviders.com](http://www.elproviders.com)



A PBM Group Company

85 Orchard Road • Skillman • New Jersey • 08558  
 800.526.2125 • 732.246.3366 • Fax: 732.246.0570 • [www.lifesignmed.com](http://www.lifesignmed.com)

Printed in USA.  
 All rights reserved.  
 80645 - 02/09/2021